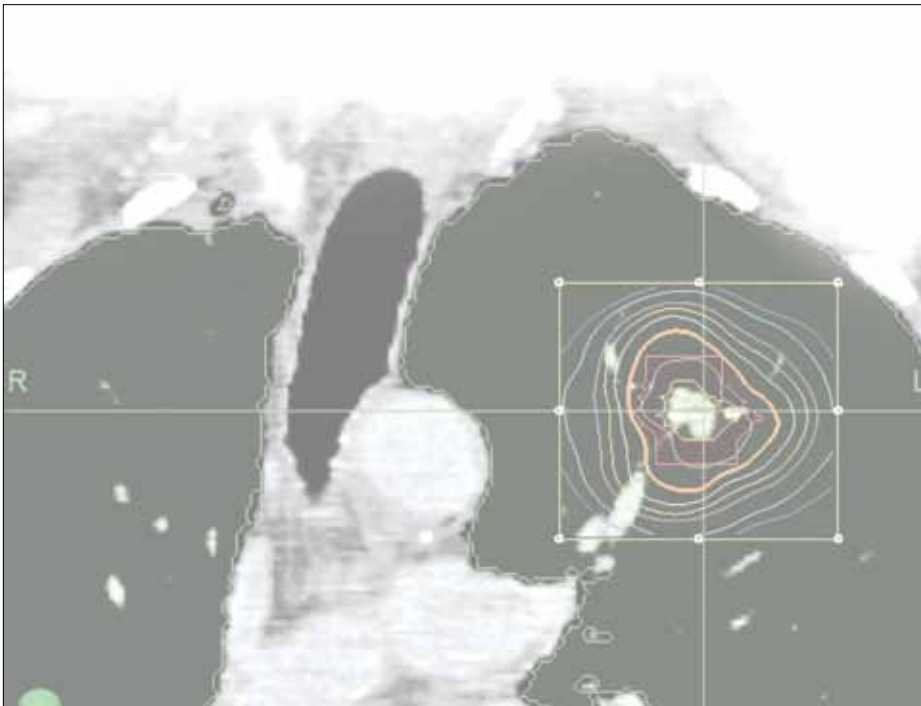


Lung Radiosurgery

Precisely Delivering Ablative Radiation
to Lung Tumors While Maximizing
Healthy Tissue Preservation

**Seattle CyberKnife Center**

James Tower
Life Sciences Building
550 17th Ave., Suite A10
Seattle, WA 98122
T 206-320-7130
F 206-320-7137

Lung cancer is the leading cause of cancer death in the United States.¹ While surgery is often considered the gold standard of treatment, its application is frequently limited by the patient's health and the stage and location of the tumor.



A common non-surgical alternative for medically inoperable patients includes external beam radiotherapy, yet poor outcomes and high complication rates have precluded widespread use.² Supported by compelling scientific evidence, hypofractionated high-dose radiosurgery is emerging as a groundbreaking treatment modality — showing evidence of improved tumor control and patient survival when compared to conventional radiotherapy.³ Safely delivering lung radiosurgery presents considerable challenges given continuous respiratory-induced motion and significant surrounding healthy tissue. As a proven solution for treating medically inoperable lung cancer patients and those seeking an alternative to surgery, the CyberKnife® Robotic Radiosurgery System stands alone in addressing these challenges — setting new standards for accuracy, conformality and respiratory motion tracking.^{4,5}

Seattle CyberKnife Center at Swedish

Seattle CyberKnife Center at Swedish Medical Center brings new hope to people dealing with cancerous and non-cancerous tumors as well as other select medical conditions. CyberKnife is the first radiosurgery system so precise it's able to target small and complex tumors and lesions anywhere in the body, including those in the brain, head and neck, spine, lung, liver, pancreas, kidney, prostate and pelvis.

Seattle CyberKnife Center Credentialed Physicians

Seattle CyberKnife Center offers potentially life-saving technology and an impressive team of CyberKnife-credentialed physicians. Led by co-directors Sandra S. Vermeulen, M.D., and Marc R. Mayberg, M.D., these 'CyberSurgeons' are experts in their fields with years of knowledge and experience. With a collaborative, team approach, they will work together to provide the best possible care for every Seattle CyberKnife Center patient.


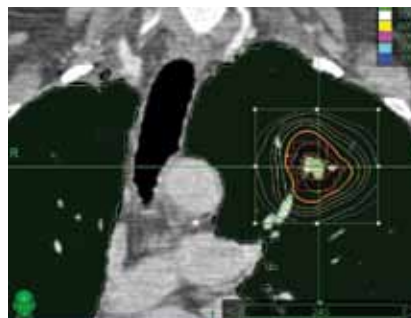


Image courtesy of Georgetown University Hospital

Improved Tumor Control and Survival

Scientific evidence has demonstrated a direct relationship between survival and efficacy with radiosurgical dose escalations of greater than 100 Gy BED (see graph).³ However, dose escalation for treating lung tumors has historically been limited by the destruction of normal tissue resulting from the large treatment margins commonplace with conventional, gantry based radiation therapy systems.

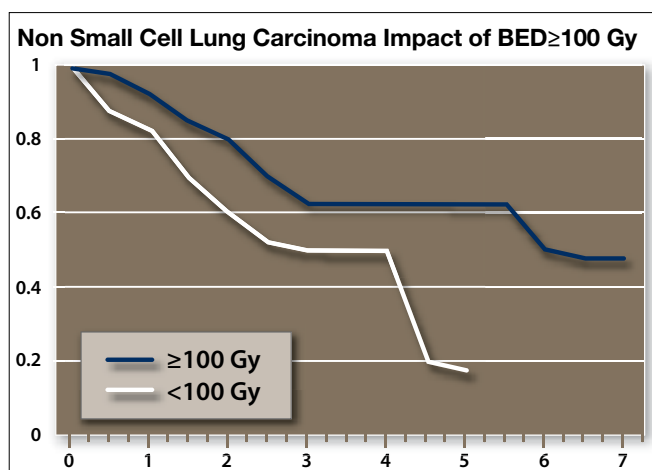

Unparalleled Healthy Tissue Preservation

Unlike the respiratory gating and breath-holding techniques of gantry systems, the CyberKnife System delivers tightly contoured beams that move precisely with tumor motion throughout the respiratory cycle. Integrating the Synchrony[®] Respiratory Tracking System, margins are limited to only those which are clinically relevant, dramatically decreasing the targeting and set-up margins typical with other respiratory compensation techniques. And with the 4D treatment planning capabilities of the CyberKnife System, tumor and surrounding structure motion and deformation are accounted for throughout the entire breathing

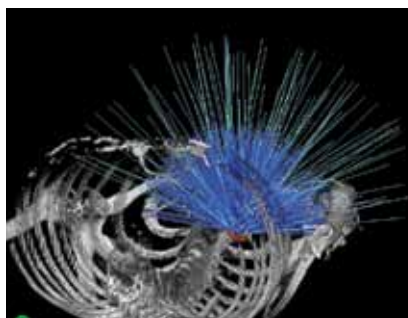


Image courtesy of Georgetown University Hospital

cycle. Ultimately, the combination of these capabilities helps to ensure prescribed doses are limited to the intended target while maximizing healthy tissue preservation.

Where conventional gantry-based imaging systems such as IGRT and cone-beam CT provide image guidance for pretreatment set-up, target movements during the treatment remain unrecognized with these technologies. Using advanced robotic technology, the CyberKnife[®] System is the only system to utilize continual image guidance to automatically track, detect and correct for target movements throughout the treatment. This intra-fraction image guidance capability unique to the CyberKnife System has proven to deliver an unprecedented 0.7 mm targeting accuracy when treating targets that move with respiration.⁶

Completely Non-Invasive Capabilities

Without the need for external frames or implanted fiducials the CyberKnife System is the only system to directly track lung tumors with radiosurgical accuracy.⁷ Using sophisticated image processing and registration, tumors can be tracked non-invasively — eliminating the risks associated with fiducial implantation while expediting the patient's time to treatment.

Unrivaled Dose Sculpting

For more than 30 years, technical research and clinical studies have demonstrated that a large array of uniquely angled beams enhances the ability to precisely sculpt delivered dose to the target while reducing the risks of surrounding tissue dose toxicity. Unconstrained by the clockwise/counter-clockwise gantry rotations of conventional radiation therapy equipment, the robotic mobility of the CyberKnife® System extends these proven benefits by delivering diverse non-coplanar treatments routinely in daily clinical practice. Using sophisticated dose calculations, a typical CyberKnife treatment includes more than 100 uniquely angled, highly focused beams per fraction — unlike the 7 to 9 beam plans commonplace with conventional IMRT treatment delivery.

Proven Capabilities — Proven Results

The CyberKnife System is now recognized as the premier solution for safe and effective lung radiosurgery delivery. With a large body of academic support, the CyberKnife System has now treated more than 50,000 patients and been installed as the radiosurgery system of choice by more than 140 institutions globally — including many of the most prestigious cancer centers in the world.

For More Information

Seattle CyberKnife Center
James Tower
Life Sciences Building
550 17th Ave., Suite A10
Seattle, WA 98122
Phone: 206-320-7130
Fax: 206-320-7137

References

1. American Cancer Society Web site: www.cancer.org.
2. Wagner H. Image-guided conformal radiation therapy planning and delivery for non-small-cell lung cancer. *Cancer Control*. 2003 Jul-Aug;10(4):277-88.
3. Onishi H, Araki T, Shirato H, Nagata Y, *et al*. Stereotactic hypofractionated high-dose irradiation for stage I nonsmall cell lung carcinoma: clinical outcomes in 245 subjects in a Japanese multiinstitutional study. *Cancer*. 2004 Oct 1;101(7):1623-31.
4. Pennathur, A., Luketich, J. D., Burton, S., *et al*. Stereotactic radiosurgery for the treatment of lung neoplasm: initial experience. *Ann.Thorac.Surg*, 83[5], 1820-1824. 2007.
5. Seppenwoolde, Y., Berbeco, R. I., Nishioka, S., *et al*. Accuracy of tumor motion compensation algorithm from a robotic respiratory tracking system: a simulation study. *Med.Phys.*, 34[7], 2774-2784. 2007.
6. Dieterich S, Taylor D, Chuang C, *et al*. The CyberKnife Synchrony Respiratory Tracking System: Evaluation of Systematic Targeting Uncertainty. Synchrony® Respiratory Tracking System clinical accuracy specification of 1.5 mm for moving targets.
7. Limited to laterally located lung tumors greater than 1.5 cm in diameter. Centrally located tumors and those smaller than 1.5 cm may require fiducial implantation. Xsight® Lung Tracking System clinical accuracy specification of 1.5 mm for moving targets.



SEATTLE CYBERKNIFE CENTER

James Tower
Life Sciences Building
550 17th Ave., Suite A10
Seattle, WA 98122
T 206-320-7130
F 206-320-7137
www.swedish.org